

# Motion– Activated Exit Reminders

# 4.14



The memo reminder uses a familiar voice as prompts.



Motion Activated Memo Reminder

## Purchasing information:

**Doability** : approx \$130 + P&P

Email: [info@doability.com.au](mailto:info@doability.com.au)

Website: [www.doability.com.au](http://www.doability.com.au)

Disclaimer: This information is a guide only. Alzheimer's WA Ltd does not endorse any of the manufacturers, suppliers or the use of any of the products featured. It is recommended that an assessment be sought from the Independent Living Centre or an Aged Care Assessment Team prior to purchasing Assistive Technology.

Revised 23/09/2020

## DESCRIPTION

The Motion-Activated Memo Reminder is a system to prompt people with dementia by using verbal messages.

The verbal message is recorded and the minder is placed at the approach to the doorway.

A timer plug can be set to operate the minder only at key times e.g. at night.

Once the system is in position, the infra red motion detector senses movement when the door is approached, triggering a short familiar pre-recorded verbal message.

## APPLICATIONS

The reminder can prompt people to remember to take their keys if they are going out or could be used to remind a person that it is evening and not to go for a walk.

Any prompting message can be recorded.

Battery-operated and easy to install, requiring no technical input.